




**Pixel**<sup>®</sup>CO<sub>2</sub><sup>™</sup>

## Fractional CO<sub>2</sub> Laser Skin Resurfacing

### Pixel CO<sub>2</sub> - Ultra Pulse Mode Protocol

Clinical Indications	On time (msec)	Off time (msec)	Pulse Duration (μs)	Frequency (Hz)	No. of Passes	No. of Treatments
Acne Scars and Facial Pores Improvement	110	600	350	300	3	3 - 5
Photo-damaged Skin Clearing	50	100	200	250	5 - 10	3 - 5
Smoothing Fine Lines, Wrinkles and Rhytids	140	500	200	275	3	3
Skin Texture Improvement	130	600	400	400	5 - 10	3 - 5

**Alma** Lasers<sup>™</sup>  
Wellbeing Through Technology<sup>®</sup>

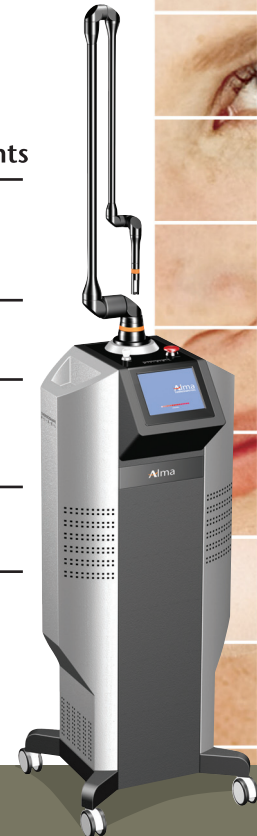

PBPX10090701 Rev.1

w w w . a l m a l a s e r s . c o m